


THE ONE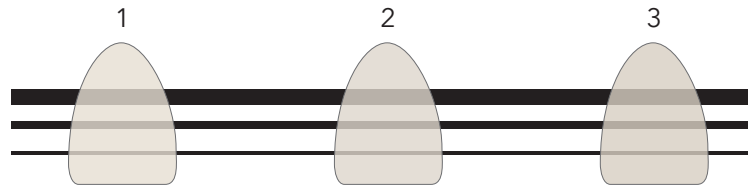


POWDER CROMA HIGH FLUORESCENCE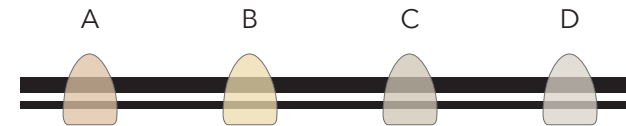


PASTE CROMA HIGH FLUORESCENCE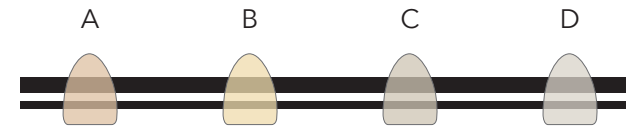


PASTE CROMA LIGHT HIGH FLUORESCENCE

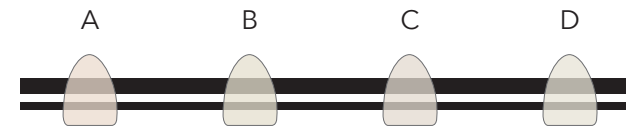
THE ONE 1

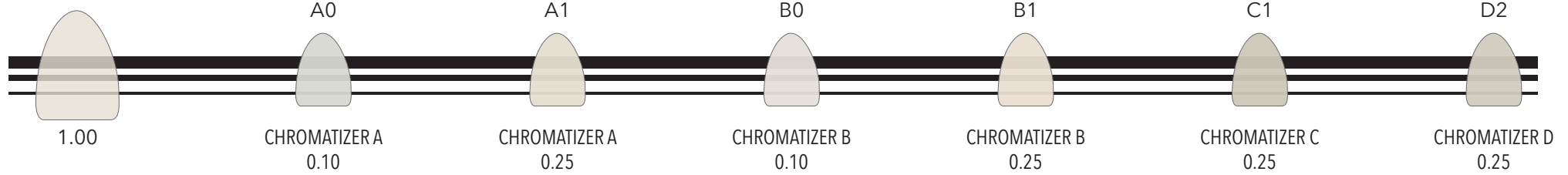THE ONE 2

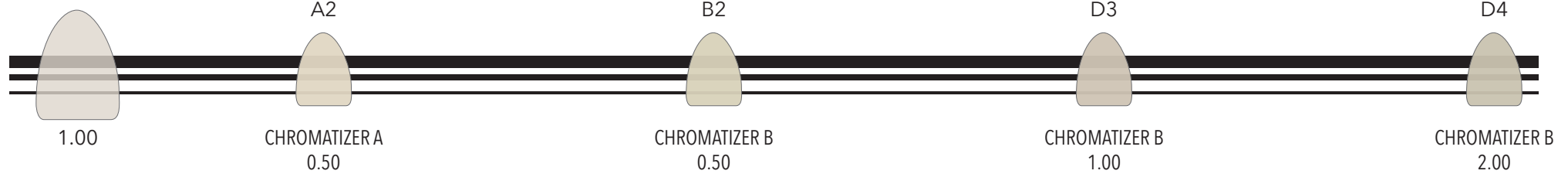THE ONE 3

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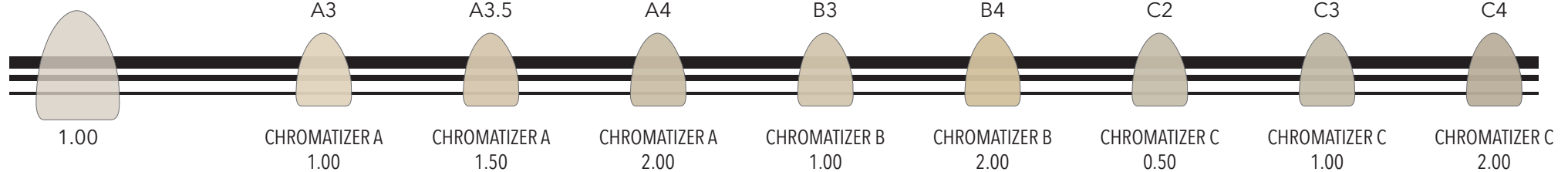PROGRAM	DRY TIME	CLOSING TIME	START TEMPERATURE	SATURATION *	HEAT RATE	FINAL TEMPERATURE**	VACUUM START	VACUUM STOP	HOLD TIME	NOTE
	min	min	°C	min	°C / min	°C	°C	°C	min	
Bond 007	1 - 2	6 - 8	450	1	80	980	450	980	6	DRY TIME and CLOSING TIME should be changed for bigger restorations or when fire multiple restorations at once
Opaque 1st & 2nd	6	6	400	1	55	950	405	950	1	After firing, opaque appearance should be lightly bright, egg like and well fired; if it appears too dull rise temperature.
SiD & Cuspid Dentin	2 - 4	6	400	1	45	920	405	920	1	Increase DRY TIME and CLOSE TIME on big restorations or if multiple restorations are fired at once.
THE ONE 1st	2 - 4	4	400	1	45	880	405	880	1	Increase DRY TIME and CLOSE TIME on big restorations or if multiple restorations are fired at once.
THE ONE 2nd	2 - 4	4	400	1	45	870	405	870	1	Increase DRY TIME and CLOSE TIME on big restorations or if multiple restorations are fired at once.
Stains Fixing	2	4	400	0	45	770	NO	NO	1	Fix stains on surface and between layered masses, this firing avoids mixes, this program can be used several time.
Stains Firing	2	4	400	1	45	800	405	800	1	After firing, stains should appear glossy and shiny, increase DRY TIME for bigger restorations.
Glaze FX	4	2	400	0	45	850	NO	NO	1	To be used only with paste GLAZE FX, suitable both for layering and pressed ceramic.

GENERAL INFORMATIONS:

All the data shown are indicative and must be modified according to specific cases (long bridges, circular etc.). A good drying of the restorations is recommended.


*** SATURATION:**

This program can only be run from ovens equipped with this function; ask your oven manufacturer for more information.

**** FINAL TEMPERATURE:**

FINAL TEMPERATURE may vary according to the size of the restoration, single crowns, bridges etc; moreover, it may also vary due to brands, models and the state of wear of the oven. For IVOCCLAR brand ovens and similar, set the VACUUM STOP 1°C lower than the FINAL TEMPERATURE, so as to carry out the HOLD TIME without vacuum.

- www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it

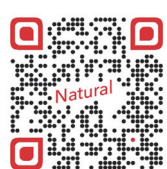Tressis Italia srl

viale Italia 194, 31015 Conegliano (TV) ITALY
 TEL (+39) 0438 41 83 16 - FAX (+39) 0438 42 64 50
 Assistenza Tecnica Online (39) 347 944 28 60
 web: www.tressis.it - email: info@tressis.it

DENTAL CERAMIC

ITALIAN STYLE

REF 51 TABHT 030-1