
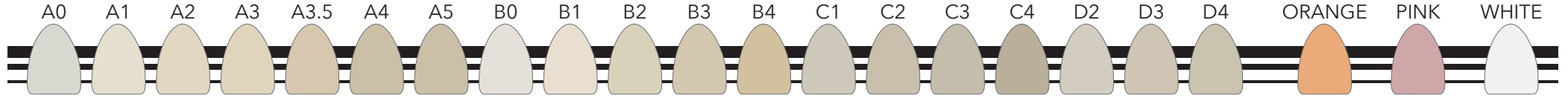
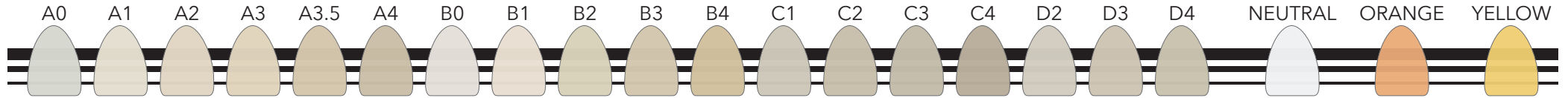
OPAQUE PASTE


SHOULDER MASS

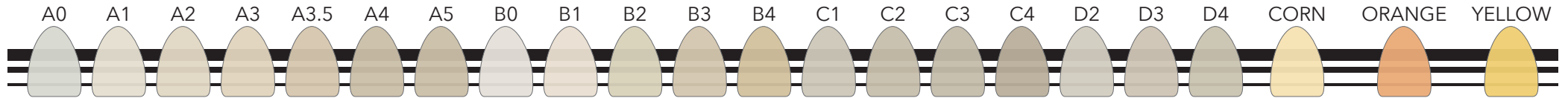OPAQUE DENTIN

SID & CUSPID DENTIN

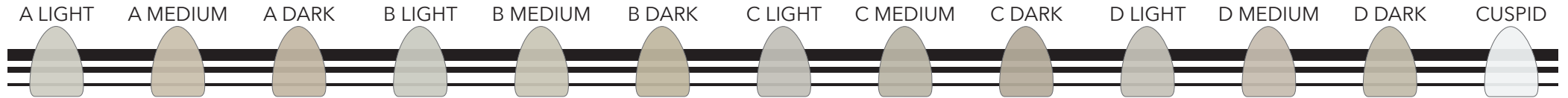BODY DENTIN

- www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it

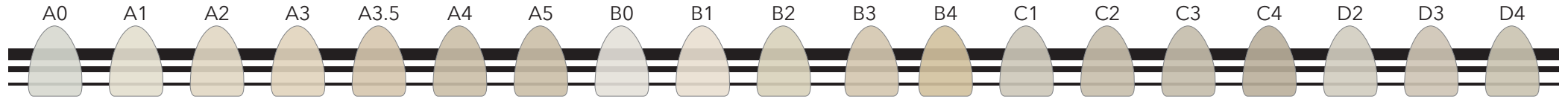
Tressis Italia srl
 viale Italia 194, 31015 Conegliano (TV) ITALY
 TEL (+39) 0438 41 83 16 - FAX (+39) 0438 42 64 50
 Assistenza Tecnica Online (39) 347 944 28 60
 web: www.tressis.it - email: info@tressis.it

DENTAL CERAMIC


 ITALIAN STYLE

REF 21 TABLF 010 -1


DENTIN


DENTIN MODIFY

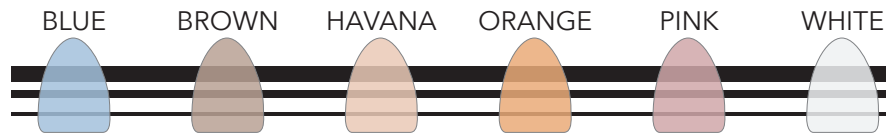GINGIVAL

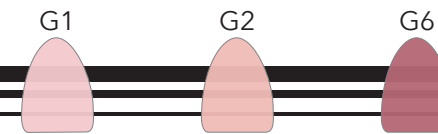CORRECTION

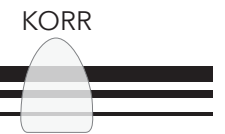
ENAMEL


ENAMEL SPECIAL

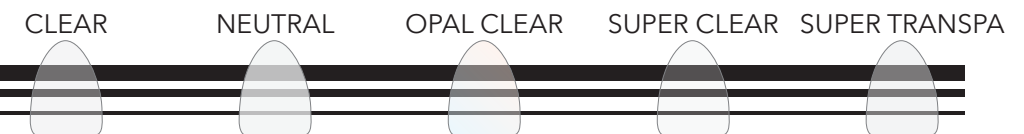TRANSPARENT

TRANSPARENT COLOR

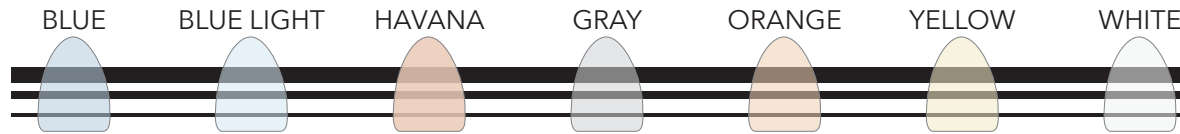TRANSPARENT OPAL

- www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it

Tressis Italia srl

viale Italia 194, 31015 Conegliano (TV) ITALY
 TEL (+39) 0438 41 83 16 - FAX (+39) 0438 42 64 50
 Assistenza Tecnica Online (39) 347 944 28 60
 web: www.tressis.it - email: info@tressis.it


DENTAL CERAMIC

 ITALIAN STYLE

REF 21 TABLF 010 -1

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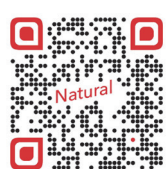 ITALIAN STYLE

REF 21 TABLF 020 -1

PROGRAM	DRY TIME	CLOSING TIME	START TEMPERATURE	SATURATION *	HEAT RATE	FINAL TEMPERATURE**	VACUUM START	VACUUM STOP	HOLD TIME	NOTE***
	min	min	°C	min	°C / min	°C	°C	°C	min	
Opaque 940°C 1st & 2nd	6	6 - 8	400	1	55	940	405	940	1	DRYTIME and CLOSINGTIME should be changed for bigger restorations or when fire multiple restorations at once
Opaque 800°C 1st & 2nd	6	6 - 8	400	1	55	800	405	800	1	After firing, opaque appearance should be lighty bright, egg like and well fired; if it appears too dull rise temperature.
Shoulder Mass 1st & 2nd	2 - 4	6	400	1	45	790	405	790	1	To mix powders use only Liquid for Shoulder Masse.
SiD & Cuspid Dentin	2 - 4	4	400	1	45	780	405	780	1	Increase DRYTIME and CLOSE TIME on big restorations or if multiple restorations are fired at once.
Dentin 1st	2 - 4	4	400	1	45	770	405	770	1	Increase DRYTIME and CLOSE TIME on big restorations or if multiple restorations are fired at once.
Dentin 2nd	2 - 4	4	400	1	45	760	405	760	1	
Correction Masse	2	4	400	1	45	730	405	730	1	Correction Masse firing program for pure usage, if mixed with other layering powders change FINAL TEMPERATURE.
Stains Fixing	2	4	400	0	45	700	NO	NO	1	Fix stains on surface and between layered masses, this firing avoids mixes, this program can be used several time.
Stains Firing	2	4	400	1	45	730	405	730	1	After firing, stains should appear glossy and shiny, increase DRY TIME for bigger restorations.
Glaze FX	4	2	400	0	45	740	NO	NO	1	To be used only with paste GLAZE FX, suitable both for layering and pressed ceramic.

GENERAL INFORMATIONS:

All the data shown are indicative and must be modified according to specific cases (long bridges, circular bridges, etc.). A good drying of the restorations is recommended.

*** SATURATION:**

This program can only be run from ovens equipped with this function; ask your oven manufacturer for more information.

**** FINAL TEMPERATURE:**

For IVOCLAR brand ovens and similar, set the VACUUM STOP 1°C lower than the FINAL TEMPERATURE, so as to carry out the HOLD TIME without vacuum.

***** NOTE:**

All the parameters in this table have been defined using an F1 DIGITAL PRESS oven; manufacturer, type, age, updating and overall status of the oven can cause the data shown here to vary, even considerably.

It is recommended to carry out a test firing before starting the actual work.

Should changes be made to the programs, it is recommended to make small changes (±10 °C) at a time.

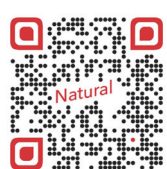
****** PRESS NOTE:**

The use of the X1 investment cylinder is never recommended, in any case, not even for single restorations.

The use of disposable pistons in refractory material is recommended.

WAX WEIGHT	PELLETS	START TEMPERATURE	HEAT RATE	FINAL TEMPERATURE	HOLD TIME	PRESS TIME	PRESSSION	PRESS NOTE****
	n°	°C	°C / min	°C	min	min	BAR	
Up to 0,6 gr.	1x2gr	700	60	940	20	6 - 8	4,5	Wax weight sprues included
From 0,6 gr to 1,4 gr.	2x2gr	700	60	940	20	8 - 10	4,5	Wax weight sprues included, ONLY 200gr cylinder
From 1,4 gr to 1,8 gr.	1x5gr	700	60	960	30	10 - 12	4,5	Wax weight sprues included, ONLY 300gr cylinder

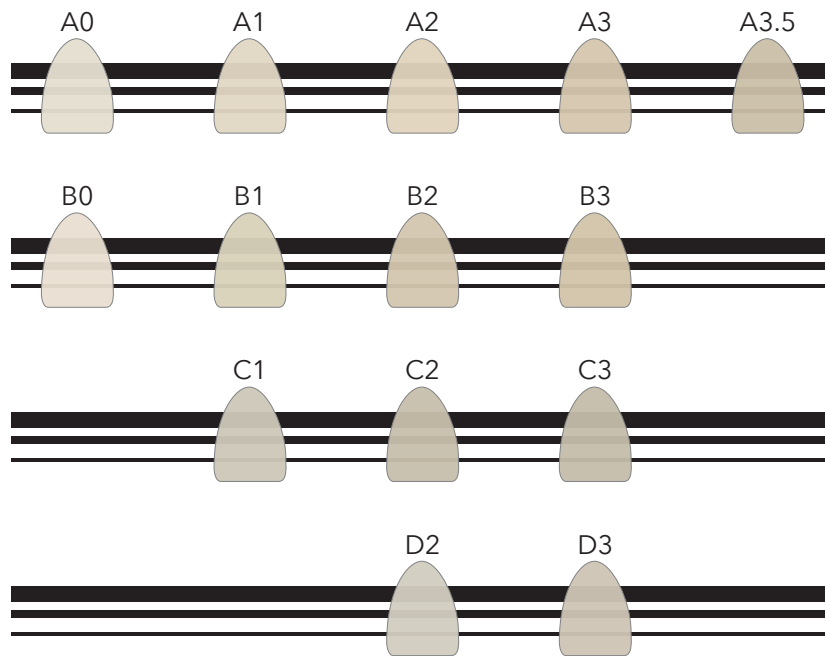
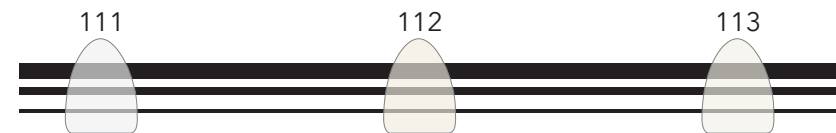
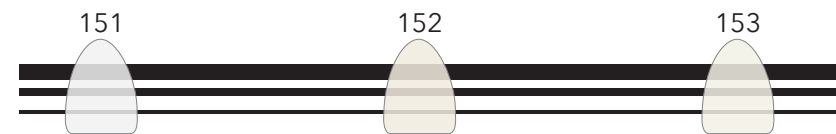
- www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it - www.naturalceramic.it

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